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MEDICAL DEVICE

Intended use

Treatment of phimosis through progressive dilation of the phimotic ring.

Description

Phimosis is a narrowing of the foreskin that prevents the man from uncovering the glans. This condition can make it difficult and very painful to fully and independently uncover the glans, it does not allow the man to live his sexuality peacefully and can have complications (e.g. paraphimosis). Phimosis can present itself with different levels of severity, as in the table below:

Type of phimosis	Severity level	Description
Not tight	Grade 1	Total retraction but tight at the base
Not tight	Grade 2	Retracted but not under the glans
Tight	Grade 3	Partial retraction
Tight	Grade 4	Slight retraction
Tight	Grade 5	No retraction

PHIMOSTOP can solve the condition of phimosis without circumcision surgery. Phimostop is composed of 22 tuboids specially designed to be positioned on the glans and perform a progressive and gentle dilatation, acting on the part affected by the shrinkage. The tuboids are made of soft, non-allergenic medical silicone and, thanks to a patented fastening system, can be worn without any problem during the day or night. Phimostop is used in cases of both acquired and congenital phimosis.

Instructions for use

1. Choose the tuboid that can be applied to the glans without excessive forcing of the foreskin and without pain
2. Wash your hands and the tuboid with soap and water before putting it on. You can apply the wet tuboid to facilitate insertion
3. Insertion of tuboids 1 to 9:
 - A. Lower the foreskin as far as possible without forcing
 - B. Insert the tuboid with a slight rotation and rest the base on the glans
 - C. You can moisten the tuboid (not the fins) with neutral lubricant
 - D. Pull up the foreskin below the wings
 - E. Ensure that the phimotic ring remains around the tuboid
 - F. Lower the flaps and fix them with the supplied plaster
4. Insertion of tuboids 10 to 22:
 - A. Lower the foreskin as far as possible without forcing
 - B. Place the base of the tuboid on the glans
 - C. You can moisten the tuboid (not the fins) with neutral lubricant
 - D. Pull up the foreskin below the wings
 - E. Ensure that the phimotic ring remains around the tuboid
 - F. Lower the flaps and fix them with the supplied plaster
5. As the phimotic ring expands, move on to the next tuboid
6. Treatment ends when the foreskin can descend to the base of the glans without squeezing and flows freely even when erect
7. The tuboids can be worn 24 hours a day but not less than 12 hours a day

Warnings

1. Wear the same tuboid for at least 2/5 days depending on the condition and state of phimosis
2. You can wear the next tuboid when it enters without pain, otherwise prolong the wearing time of the current tuboid
3. Avoid using more than necessary sizes immediately
4. To urinate it is not necessary to remove the tuboid
5. Wear briefs when wearing the tuboid
6. Use as many tuboids as necessary without skipping steps
7. It will not always be necessary to use the final tuboids. When the foreskin goes down without restriction, the therapy is finished
8. With the tuboid inserted, any activity is possible
9. After use and before reusing the tuboids, it is recommended to clean them with soap and water to maintain an adequate level of hygiene
10. Do not use the tuboid in case of lacerations
11. In case of a serious accident in connection with the use of the device, it is recommended to report it to the manufacturer and the Competent Authority

Contraindications

Phimostop has no side effects and does not cause pain. In the occurrence of inflammation or pain/redness, it is recommended to discontinue the protocol and contact your doctor and our customer service.

Do not use in case of:

- Paraphimosis
- Lichen sclerosus atrophicus
- Untreated lesions and/or infections
- Hypersensitivity to the material of which the tuboids are composed

Composition

The tuboids are made of hypoallergenic silicone

Packaging

The package contains 22 tuboids from 6 mm up to 42,75 mm in diameter. Included in the package is a plaster on a reel, Medical Device CE, Manufacturer EUROSIREL S.p.A., Viale Europa 30 - 20047 Cusago (MI)

Storage

It is recommended to keep the device away from heat sources

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